



DreamWisp **patient fitting guide**

Follow the instructions to get started fitting your DreamWisp mask

1	 A close-up of a person's nose with the mask's cushion being fitted over it. The person is holding the mask with their finger.	Select the cushion that fits the width of your nose without blocking your nostrils. The top of the cushion may sit between the bridge and the tip of your nose. There are five cushion sizes available: petite, small, medium, large, extra-large
2	 A close-up of hands unclipping a magnetic clip from the headgear.	Unclasp one or both of the magnetic clips from the magnetic clip receiver.
3	 A person pulling the headgear over their head.	Pull the headgear over the top of your head.
4	 A person placing the cushion over their nose.	Place the cushion over your nose.
5	 A person clasp the magnetic clips to the magnetic clips receivers.	Clasp the magnetic clips to the magnetic clips receivers.
6	 A person adjusting the headgear straps.	To adjust the headgear, peel the bottom headgear tabs away from the fabric. Evenly adjust the length of the straps. Press the tabs back against the fabric to reattach. Repeat this step with the top straps.
7	 A person positioning the mask on their face.	Position the mask until it fits comfortably. When finished, the elbow should rest at the top of the head. The cushion should rest comfortably on your face without blocking your nostrils. <small>Note: Please read the Instructions for use for further fitting information.</small>